iHealth™

Wireless Pulse Oximeter

## QUICK START GUIDE

### SET-UP INSTRUCTIONS

#### CHARGE THE BATTERY

Plug the iHealth Wireless Pulse Oximeter PO3 into a USB port for three hours or until the battery indicator turns off.

#### DOWNLOAD APP

Download and install the app from the App Store. Search for "iHealth SpO2". (Your compatible iOS device should be version 5.0 or later.)

iHealth SpO2

#### CREATE USER AND CLOUD ACCOUNT

After downloading the app, register and set up your user account following on-screen instructions. Upon setting up your user account, you will also have access to a free, secure iHealth Cloud account. Go to [www.ihealthlabs.com](http://www.ihealthlabs.com), then click "Sign In" to access your cloud account from PC or Mac.

### TURN BLUETOOTH "ON"

Turn Bluetooth "On" under the "Settings" menu of your iOS device. (The date and time of the Pulse Oximeter will be synced with your iOS device upon first successful connection.) Once your Bluetooth is on, the Pulse Oximeter will connect automatically when the app is launched.

### TESTING INSTRUCTIONS

1. Open the clamp of the Pulse Oximeter PO3, then place your middle, ring or index finger of your left hand into the rubber opening of the oximeter with nail side down, as pictured.

2. On the front panel, press the "Start" button once to turn the oximeter on.
3. Keep your hand still for the reading.
4. After a few seconds, your SpO2 reading will appear on the oximeter display screen and the app if the app is turned on.
5. If the signal strength is too low, switch to another finger and perform the test again.

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